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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Primo 4 wheels rollator is a walking aid for people with reduced mobility. This rollator is designed for indoor and outdoor use and folds up for easy storage or transport.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 130 Kg.
- Do not exceed the maximum load limit in the basket (5kg). Only transport loads in the basket. Do not hang heavy objects on the rollator as this could affect its stability.
- Make sure that all parts (wheels, cables, handles and knobs) are secure and in good working order.
- Do not use the rollator on stairs or escalators.
- Only use accessories and parts supplied by the manufacturer.
- The rollator is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may damage the wheels and the structure of the rollator itself. The rollator could be weakened and no longer meet its mechanical characteristics in future use. Only use the rollator as a walking support.
- Do not use the rollator to walk backwards.
- Do not make any adjustments to the rollator while it is in use.
- Check that the brakes work properly before use.
- All wheels must always be in contact with the ground.
- When the rollator is in the stationary position, the brakes must be locked. Do not attempt to move the rollator if the brakes are locked.
- Be sure to put your hands on each handle so that the rollator can be in the parallel force position.
- Use caution when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Do not use the walker on a wet, icy, or snowy slope.
- If any parts are damaged, broken or worn, contact the dealer.

PRODUCT DESCRIPTION

1 : Handgrip (x2)
2 : Brake handle (x2)
3 : Plate (x1)
4 : Brake cable (x2)
5 : Basket (x1)
6 : Adjustment knob (x2)
7 : Rear wheel (x2)
8 : Plastic net with cane holder (x1)
9 : Front wheel (x2)

MOUNTING & USE

UNFOLDING :

Place the rollator on its wheels. You can unfold the chassis frame by pulling one of the rear wheel tubes (1). Push the cross brace straight down (2) until it easily clicks into place. Rotate the movable folding guard (3) as far as it will go (3) so that it is vertical to the cross brace.

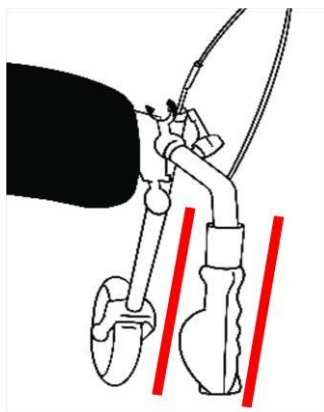
ASSEMBLY OF ACCESSORIES :

Hang the basket on the small hooks in the front underneath the seat and place the tray down with the small feet in the small holes in the seat section.

HANDLE HEIGHT ADJUSTMENT :

Insert push handles at least in the frame tube. Secure them at the desired height by turning the locking lever clockwise. Adjust both handles to the same height. The push handles must be adjusted parallel to the direction of movement! Maximum permissible angle between the longitudinal center line of the handle and the direction of movement is 0°.

The position of the adjustment knob itself can be adjusted by simultaneously pulling and rotating it. To change the height setting of the push handles, rotate the adjustment knob counterclockwise in order to release the fixation. Then lock the handle at the desired height, as described above.

BRAKE ADJUSTMENT :

If braking is poor or too hard, the brake can be adjusted either at the brake housing above or using the adjusting screw on the Bowden cable below to achieve the desired braking effect.

The brake is correctly adjusted when the brake levers as well as the brake wires are free of play and the wheels can be rotated without grinding noises when the brake is released.

USE OF BRAKE HANDLES :

To brake when walking with the rollator, pull both break handles upwards with your hands. Always brake slowly while applying equal pressure with both hands.

To apply the brakes/park the rollator, press both break handles downwards until they audibly click into place! To release the brake, pull both brake handles upwards once again until they are released (brake handles are in the middle position). The braking effect can be impaired by wear and tear on the tires. For this reason, regularly check the braking effect.

SITTING POSITION :

Place the rollator on an even and solid surface. Then apply the brakes. The rollator must be stable and may not slip. Only sit upright; do not lean backwards!

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.

- **Checklist :**

To keep the rollator in good condition, check before each use, and check regularly as follows :

Check the item	Anytime	Weekly	Monthly	Yearly
Clip and knob for height adjustment		X		
Structural weld at wheels		X		
Cleanliness	X			
Screws and fasteners		X		
Stability of frame and backrest			X	
Condition of seat				X
Brakes	X			

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
 - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width 59 x Depth 72 x Height 80 / 93 cm
Seat Dimensions	Width 38 x Depth 17,5 x Height 59 cm
Basket Dimensions	Width 40 x Depth 27 x Height 16 cm
Plate Dimensions	Width 39 x Depth 25,5 cm
Maximum basket load	5 kg
Maximum plate load	5 kg
Weight	9 kg
Maximum supported weight	130 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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