**ROLLATOR APPUIS ANTE - BRACHIAUX NEO
SUPPORT
ROLLATOR ANTE SUPPORTS - BRACHIAUX NEO
SUPPORT
REF. 826242**

MANUEL D'INSTRUCTIONS - USER MANUAL

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This manual presents the instructions for the classic handle version. The Neo Support version with anterior brachial supports is identical. The specific parts are described at the end of the manual.

For your safety, please read this instruction manual before using the rollator.

ROLLATOR NEO SUPPORT – ROLLATOR NEO SUPPORT

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The Rollator Neo Support is designed for indoor and outdoor use with a lightweight aluminum frame and 20 cm diameter wheels for easy operation on smooth surfaces. Molded handles with soft touch brakes are ideal for people with limited strength or mobility. The height of the handle can be easily adjusted from 80cm to 93cm.

This product is supplied with a removable bag that can be used with or without a shoulder strap. It has an optional cane holder.

Due to its compact mechanism, the NEO SUPPORT ROLLATOR can be transported and stored very easily.

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Carefully remove all components from the box and check the items in the list below. Any damaged or missing components should be reported to your authorized dealer or manufacturer.

Before use, remove all plastic packaging and inspect the rollator for damage, paying particular attention to the wheels, seat frame handles, seat back, and backrest, and check for missing or loose screws or bolts.

PRECAUTIONS PARTICULIERES DE SECURITE - VERIFICATIONS PREALABLES - **SPECIAL SAFETY PRECAUTIONS - DUE DILIGENCE**

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- Make sure the Rollator is fully extended and locked before use.
- Check the brakes before use.
- Make sure that all parts are securely locked and all removable parts are in good working order.

PRECAUTIONS PARTICULIERES DE SECURITE - PENDANT L'UTILISATION - **SPECIAL SAFETY PRECAUTIONS - DURING USE**

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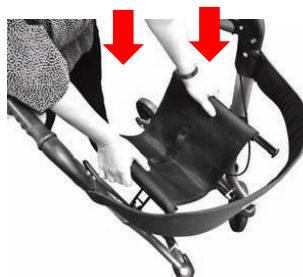
Unfold the Rollator

Remove the protective coating from all four wheels. Remove the cardboard between the rear wheels and the brake latches.

Place the walker on a stable surface.

Unfold by spreading the frame apart while pushing down, making sure each latch locks properly.

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Adjust the height of the handles by pressing the adjustment knob and sliding the tube up or down. The height will adjust one hole at a time. Make sure that both handles are adjusted to the same height.

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A series of holes are placed every 2.5 along the handle tube.
They allow the handle height to be adjusted to suit the user. The ideal handle height is set when the top of the handle is aligned with the user's wrist

INSTALLATION SUITE - INSTALLATION CONTINUED

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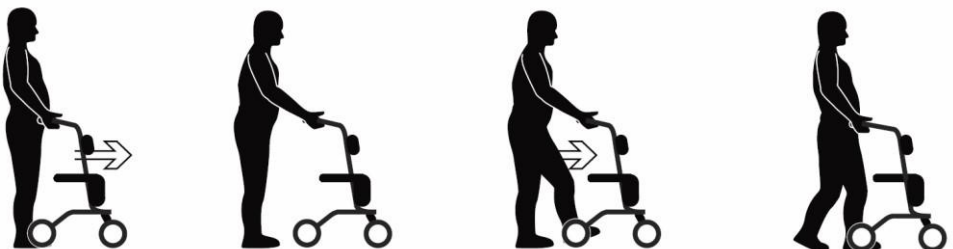
Check that the brake cables are running on the outside of the frame and adjust the cable guides to store the cables carefully against the frame.
Make sure the brakes are working properly before using the rollator.

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When using the rollator to move around, it is recommended that both hands be placed firmly on the handles, with fingers wrapped around the brake levers so that the walker can stop immediately in an emergency.
The ideal walking technique is to push the walker forward at a comfortable arm's length.

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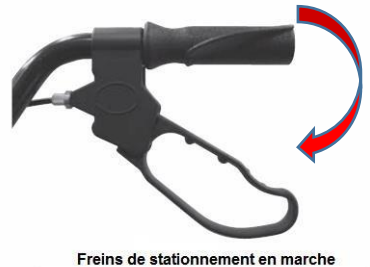
To stop the rollator, gently squeeze the 2 brake levers until it comes to a complete stop.

Brake Lock (Parking Brake Function) When using the rollator as a booster seat, the brakes must be locked.

To lock the brakes, depress the brake levers until they click. Always verify that the brake lock is activated before sitting in the seat.

To release the brake, pull the brake lever until the lock disengages. Never release the brake lock when sitting on the seat.

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Before using the rollator as a booster seat, make sure it is parked on a safe, level surface. Before sitting down, lock both brake levers to prevent possible injury.

TRANSPORT ET RANGEMENT – TRANSPORTATION AND STORAGE

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To fold the rollator, pull firmly on the strap on the seat. This will disengage the side braces and collapse the frame. Push the frame together for a compact fold.

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After unfolding the rollator, fasten the belt before use. Make sure the belt is taut and tight. When folding the rollator, you must loosen the belt.

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GENERAL SAFETY PRECAUTIONS

Do not use the rollator to carry a seated person.

A risk assessment should always be performed to ensure that the product is functioning correctly.

- Before use, the user must read these instructions. Make sure all components are correctly and securely installed.
- Make sure the user has sufficient strength to maneuver and control the rollator and brakes.
- Before sitting down, make sure the brakes are firmly locked. Do not release the brakes until the user is in a standing position.
- Avoid storing the rollator in rooms that are exposed to direct sunlight or very hot as this can cause skin injuries.
- Do not place more than 4.5 kg in the carry bag.
- When sitting down, do not try to reach objects by leaning forward. Do not lean backwards as this may cause the rollator to overturn.
- Use only on walkways or paths. Do not use on roads, streets, or highways.
- Do not use the rollator on a slope greater than 9 degrees, or on a wet, icy, or snowy slope.
- Use with caution when crossing curbs or obstacles as this could cause the rollator to tip over which could result in personal injury or damage to the rollator.

REGULAR MAINTENANCE

An annual inspection by a professional is recommended. Any service or repair, especially of brakes, should be carried out by a professional. Unauthorized repairs may invalidate the guarantee.

- Check the rollator regularly, paying particular attention to the operation of the brakes and wheels. If the wheels of the rollator are damaged in any way, stop using the rollator immediately and contact the authorized dealer or manufacturer.
- Check that the brake levers are solid and if they are loose they must be replaced before use.
- Check the efficiency of the brake system. If you notice an unknown braking sensation or any abnormality, stop using the rollator immediately and contact your authorized dealer or manufacturer.
- Check that all screws and bolts are securely installed ;

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Attention! Do not use high-pressure cleaners, steam cleaners or corrosive chemicals for cleaning.

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FRAMEWORK

Wipe the rollator frame with a damp cloth.
For more stubborn dirt, use a mild detergent solution up to 40°C. Dry well before use to avoid corrosion.
Clean the wheels with a damp brush with plastic bristles (do not use a wire brush or you may damage the rubber or painted coating).

SADDLEBAG AND SITTING

Regularly clean the upholstery materials with a mild detergent solution up to 40° C, or wash with disinfectant alcohol if necessary.
Dry well before use.

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The rollator can be reused. It must first be cleaned and disinfected according to hygienic guidelines. When transferring the rollator, remember to provide the new user with all the necessary technical documents. The rollator must first be inspected by an authorized specialist.

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