 **ROLLATOR NEO FOLD
EVOLUTION**

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DISPOSITIF MEDICAL

 **ROLLATOR NEO FOLD
EVOLUTION**

Instructions manual

MEDICAL DEVICE

 **ROLLATOR NEO FOLD
EVOLUTION**

Manual de instrucciones

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REF. 826024.CHAMPAGNE - 826024.GRIS -
826024.BLANC

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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary. User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result. Consult your doctor or health care professional before using a walking aid.

He/she will advise and inform you on the most appropriate walking technique for you. Distributor: this leaflet must be given to the users of this product.

PRODUCT USE

The Neo Fold Evolution Rollators are walking aids for people with reduced mobility. These products are intended for use indoors and outdoors and can be folded up for easy storage or transport.

DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 136 Kg.
- Do not exceed the maximum load of 5 kg in the basket. Loads must only be transported in the basket. Do not hang heavy objects on the rollator as this could affect its stability.
- Do not use the rollator on stairs or escalators.
- Use only accessories and parts supplied by the manufacturer.
- Make sure that the knobs are tightened to the same level and the handles are securely fastened.
- The rollator is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may damage the wheels and the structure of the rollator itself. The rollator could be weakened and no longer meet its mechanical characteristics in future use. Only use the rollator as a walking support.
- Do not use the rollator to walk backwards.
- Always sit on the seat when the rollator is stationary and the brakes are locked. Serious injury to the user and/or damage to the rollator frame and/or wheels can result from inappropriate use.
- Make sure all parts (wheels, cables, handles, brakes) are secure and in good working order.
- Do not make any adjustments to the rollator while it is in use.

- All wheels must always be in contact with the ground.
- Check that the brakes function properly before use.
- Make sure that the parking brakes are securely attached.
- When the rollator is in the stationary position, the brakes must be locked. Do not attempt to move the rollator if the brakes are locked.
- Be sure to put your hands on each handle so that the rollator can be in the parallel force position. Make sure the seat is clipped flat before use.
- Make sure the seat tube caps are present.
- During use, lock the parking brakes for all actions.
- Do not attempt to push the rollator while someone is sitting in the seat.
- Do not use the seat to carry objects.
- Do not use the seat without the backrest.
- Do not sit in the seat when the rollator is on a slope.
- The brakes must be in the locked position before sitting in the seat.
- Do not attempt to reach objects while sitting down by leaning forward. Do not lean backwards as this can cause the rollator to tip over.
- Be careful when walking up pavements or on inclined, uneven, or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Do not use the rollator on a wet, frozen, or snow-covered slopes. If any parts are damaged, broken or worn, contact the dealer.

ASSEMBLING

Note: The Neo Fold Evolution Rollator is supplied fully assembled and in the folded position (as shown below).


Unfolding the rollator

Carefully check all parts before installing the product. Do not assemble if any of the parts are damaged. Do not assemble if any of the items in the contents list are missing. When unpacking, make sure that the brake handles and cables are not twisted as this may affect the operation of the brakes. Place the rollator on a flat floor and unfold it by lifting the handles upwards.

Step 1 : The oval plastic main support and the frame tube must be fully assembled without leaving any gaps when locking the levers. Do not press the levers too hard as this may damage the product.

Step 2 : Tilt the locking lever towards the front of the rollator (see opposite). If resistance is felt during this operation, check that the handle adjustment tube is positioned vertically and that the folding base is correctly positioned on the rollator frame.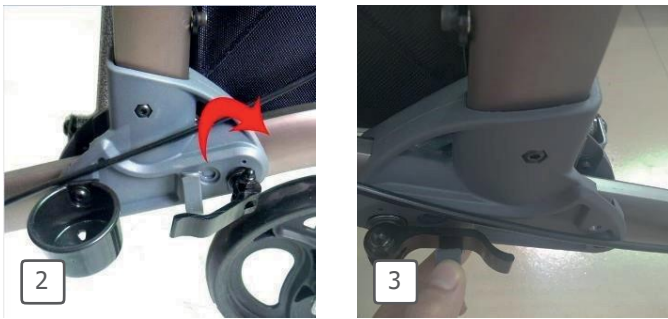

Step 3 : To lock the rollator, tilt the locking handle toward the back of the rollator, then lock the handle by placing it in the plastic slot in the base of the frame. Repeat for the other side of the rollator.

Step 4 : Remove the clamps from the folding seat

Step 5 : Press down on both sides of the folding seat to fully open the rollator.

Note : Before using the rollator, please check the following points:

Handle adjustment

The handles are adjustable in height from 82 to 97.7 cm from the floor. To ensure that the rollator is at the right height, stand behind the rollator with your arms at your sides.

Then adjust the height of the handles to the height of the user's wrists. In case of doubt, consult your dealer.


NOTE : Do not remove the inner tube from the outer tube when adjusting the height of the handles.

- For the rollator to be at the right height, stand behind the rollator with your arms at your sides. Then adjust the height of the handles to the height of the user's wrists.
- To adjust the height, unlock the knob and slide the inner tube to the desired height. Once the height has been adjusted, tighten the fixing knob again.

Satchel

Position the removable bag on its support located on the front of the rollator, as shown below.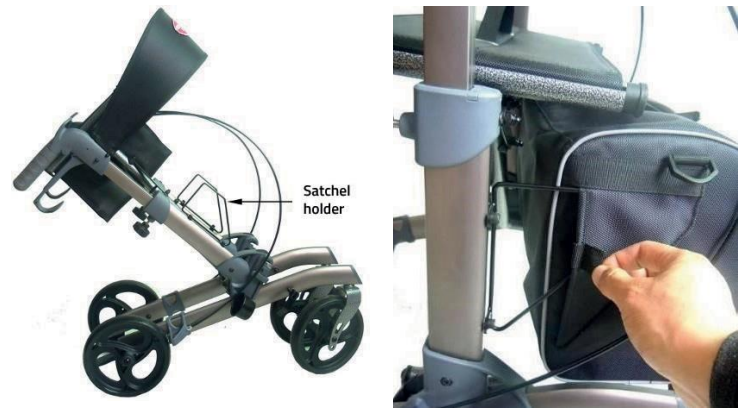


Attach the bag to the rollator frame with the velcro straps.

USE

Breaks


- To brake, push the brake handles and pull upwards (a).
- To set the parking brakes push both handles down until they lock into position. Attention, in order to be in the parking function, both handles must be locked at the same time (b).
- Raise both handles to remove the parking brakes.
- If the brakes are in operation but the rollator wheels are still moving, cease use immediately. Do not use the rollator until the brakes have been checked and properly adjusted.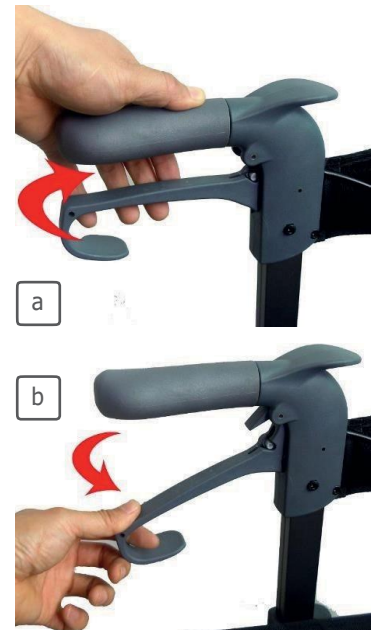

Folding the rollator

- Pull the handle on the seat upwards
- Unlock the locking handle on both sides of the rollator as described in Section 5.1 to release the frame. Then lower the handles to the folded position as below.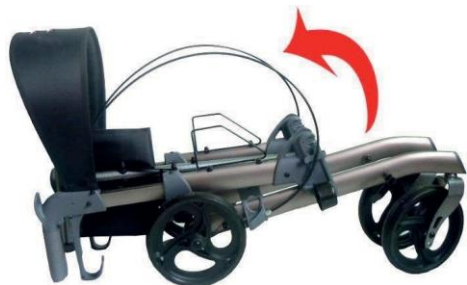

Caution : Do not loosen the locking lever during use.

Brake adjustment

- For correct adjustment, the brake mechanism, when not in use, must not touch the wheel, but the gap must not exceed 3mm. Always test the rollator after adjusting the brakes.
- If the adjustment does not apply the brakes, do not use the rollator. Contact an authorized dealer.
- To apply the brake (reduce the gap between the wheel and the brake), turn the adjusting nut clockwise.
- To release the brake (increase the distance between the wheel and the brake), turn the adjusting nut counterclockwise.

CARE AND MAINTENANCE

! Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts.
- Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized distributor.
- Check weekly the clip and knob for height adjustment and the seat

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

! Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your rollator for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Observe the conditions for storing and stowing your rollators :
 - In a dry and temperate place
 - Protect your rollator by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
 - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
 - All components must be stored without any load (do not place any objects that are too heavy on the rollator elements, do not jam anything...)

TECHNICAL SPECIFICATIONS

Overall dimensions	Width 62 x Length 68 cm x height 80 to 95 cm in 7 indexed positions
Passage width	44 cm
Handle height	82 to 97.7 cm
Seating dimensions	Width 44 x depth 23 x height 55 cm
Wheel diameter	25cm in front and 20cm at the back
Colour	Pearl White, Champagne and Grey
Aluminium structure	
Weight	7.9 kg with bag
Maximum load	136 kg

PERIOD OF USE

The expected life span of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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