 **IDENTITÉS**

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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Neo Indoor rollator is a walking aid for people with reduced mobility. This rollator is designed for indoor use and folds up for easy storage or transport. It can be used outdoors on flat surfaces.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 100 Kg.
- Do not exceed the maximum load limit in the basket (5kg). Only transport loads in the basket. Do not hang heavy objects on the rollator as this could affect its stability.
- Make sure that all parts (wheels, cables and handles) are secure and in good working order.
- Do not use the rollator on stairs or escalators.
- Only use accessories and parts supplied by the manufacturer.
- The rollator is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may damage the wheels and the structure of the rollator itself. The rollator could be weakened and no longer meet its mechanical characteristics in future use. Only use the rollator as a walking support.
- Do not use the rollator to walk backwards.
- Do not make any adjustments to the rollator while it is in use.
- Check that the brakes work properly before use.
- All wheels must always be in contact with the ground.
- When the rollator is in the stationary position, the brakes must be locked. Do not attempt to move the rollator if the brakes are locked.
- Be sure to put your hands on each handle so that the rollator can be in the parallel force position.

- Use caution when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Do not use the walker on a wet, icy, or snowy slope.
- If any parts are damaged, broken or worn, contact the dealer.

MOUNTING & USE

UNFOLDING :

1. Take the rollator out of the box and place it on the floor in an upright position. Hold the handle with one hand and grab the front of the tray with your other hand, pull the tray in the direction shown in **figure 1**. The tray is in the correct position when you hear the "click" sound.
2. Hold the handle with one hand and press the tray with your other hand in the direction shown in **figure 2**. The rollator is ready for use when you hear the "click" sound.

Figure 1


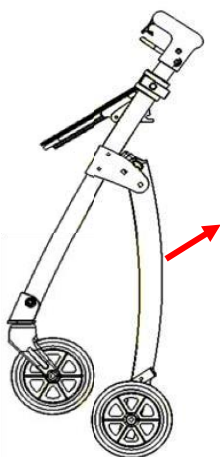
Figure 2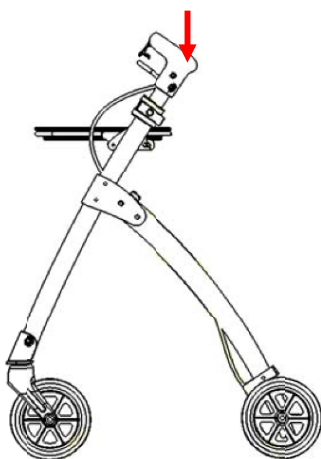

FOLDING :

1. Grab the folding bar, as shown in **figure 3**, to detach the tray. Hold the handle with one hand and the folding lever with the other. Then lift the folding bar and the rollator will fold. See **figure 4**.

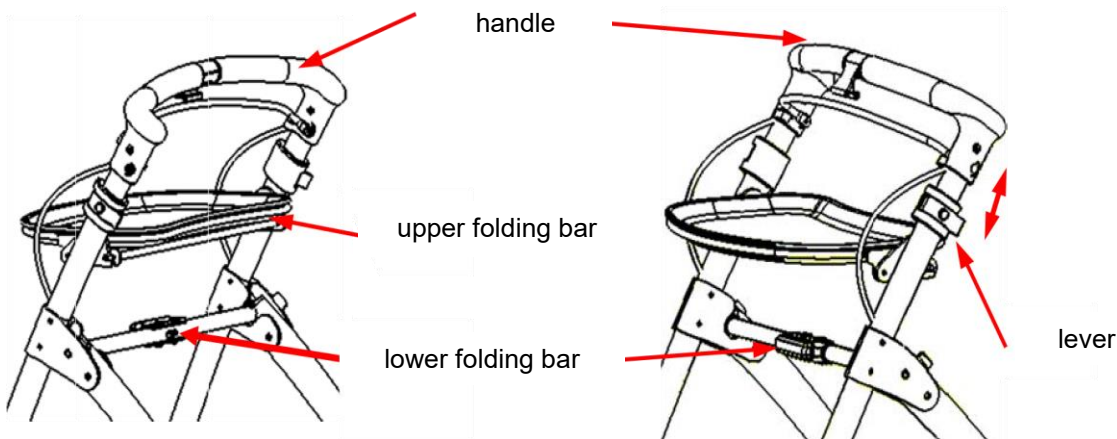
Figure 3

Figure 4

ADJUSTMENT OF THE HANDLE :

Loosen the lever and move the handle bar up or down to the desired height. As shown in **Figure 4**, insert the lever into the hole.

USE OF THE BRAKES :

1. Grab both handles and the brake bar and hold the brake bar. As shown in **figure 5**, attach the brake bar. As shown in **figure 6**, when you hear the "click" sound, the brakes are locked.
2. To release the brakes, turn the lever in the opposite direction. When you hear a "click" sound, the brakes are released.

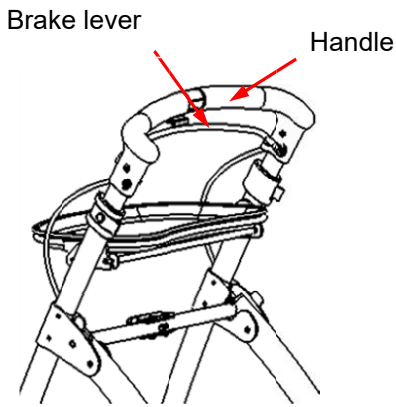
Figure 5

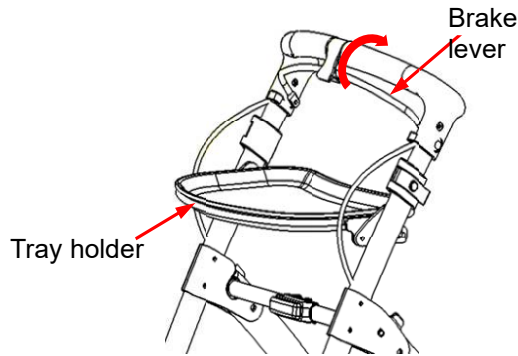
Figure 6

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.

- **Checklist :**

Check the item	Anytime	Weekly	Monthly	Yearly
Clip and knob for height adjustment		X		
Structural weld at wheels		X		
Cleanliness	X			
Screws and fasteners		X		
Stability of frame and backrest			X	
Condition of seat				X
Brakes	X			

To keep the rollator in good

condition, check before each use, and check regularly as follows :

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
 - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Larg.54 x Prof. 66 x Haut. 83 / 94.5 cm
Tray width	39 cm
Maximum tray load	5 kg
Maximum basket load	5 kg
Maximum supported weight	100 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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