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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

Victor Transfer Rollators are walking aids for people with reduced mobility. These products are designed for use indoors and outdoors, and fold up for easy storage or transport.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 110kg.

Do not exceed the maximum load limit in the basket (5kg). Loads must only be carried in the basket. Do not attach heavy objects to the rollator as this could affect its stability.

Make sure that all parts are securely fastened and in good working order.

Check the rollator's screws and fastenings regularly and retighten if necessary.

Make sure that the locking latch on the handles is properly engaged.

Make sure that both handles are tight and at the same height. Test the stability of the rollator before applying the full weight to the handles.

Do not use the seat when the rollator is on sloping or uneven ground. Only use the rollator on flat, solid surfaces.

Replace the wheels if they are badly worn or damaged.

Do not use the rollator when the footrest is extended; it must be retracted.

When the locks are released, the wheels must be able to turn freely.

Always lock the brake lever before sitting down.

Replace tyres if they are badly worn or damaged.

Do not use the rollator on stairs or escalators.

Only use the accessories and parts supplied by the manufacturer.

Do not make any adjustments to the rollator while it is in use.

All wheels must always be in contact with the ground.

Take care when climbing kerbs or using sloping, uneven or slippery surfaces. Failure to do so may result in a serious risk of falling or injury.

Do not use the walker on wet, icy or snowy slopes.

DO NOT LEAN on the footrest to sit down; the footrest must be pulled out once the patient is seated.

Check that the backrest is in the seated position and that the clips are fully extended.

If any parts are damaged, broken or worn, contact your dealer.

MOUNTING & USE

Assembly instructions

1. Remove the Rollator and all its components from the packaging.
2. Open the Rollator and push the seat down, ensuring that the side hinge bars are in the open and locked position.
3. Install the backrest support bar by inserting each end into the slots provided. It will click into place.
4. Adjust the height of the handles. The heights must be the same on the right and left. Secure the tubes by tightening the screws on each side. Make sure that the cables remain outside the structure and that the sheaths are not crushed or bent.
5. The correct height for the handles is when you are standing upright with your hands on the handles without bending your wrists.

Using the Victor Rollator

1. To slow down, lightly squeeze the handle levers with your fingers. The levers should be squeezed simultaneously on the right and left.
2. To lock the brakes, for example to immobilise the Rollator, push the 2 handle levers firmly downwards until they lock by themselves.
3. To unlock the brakes, pull up the 2 handle levers.
4. To use for transporting people, install the backrest support bar the other way round and lower the footplate.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.
- **Checklist :**
To keep the rollator in good condition, check before each use, and check regularly as follows :

Check the item	Anytime	Weekly	Monthly
Clip and knob for height adjustment		X	
Structural weld at wheels		X	
Cleanliness	X		
Screws and fasteners		X	
Stability of frame and backrest			X
Condition of seat			X
Brakes	X		

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your rollator for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your rollators.

- In a dry and temperate place

- Protect your rollator by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the rollator elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

Overall dimensions	Width 60 cm x Depth 65 cm x Handle height 78/89 cm in 6 positions
Seat dimensions	Width 35 cm x Depth 32 cm x Height 52 cm
Distance between armrests	45,5 cm
Weight	8.8 kg
Maximum load	110 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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