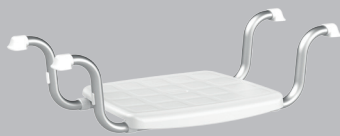
KING BATHTUB SEATS

WITH/WITHOUT BACKREST

EN USER MANUAL
 DE BENUTZERHANDBUCH
 FR MANUEL DE L'UTILISATEUR
 ES MANUAL DE USUARIO
 IT MANUALE D'USO
 PT MANUAL DO USUÁRIO
 SL NAVODILA ZA UPORABO
 HR KORISNIČKI PRIRUČNIK
 HU HASZNÁLATI UTASÍTÁS
 RO MANUAL DE UTILIZARE

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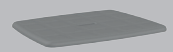
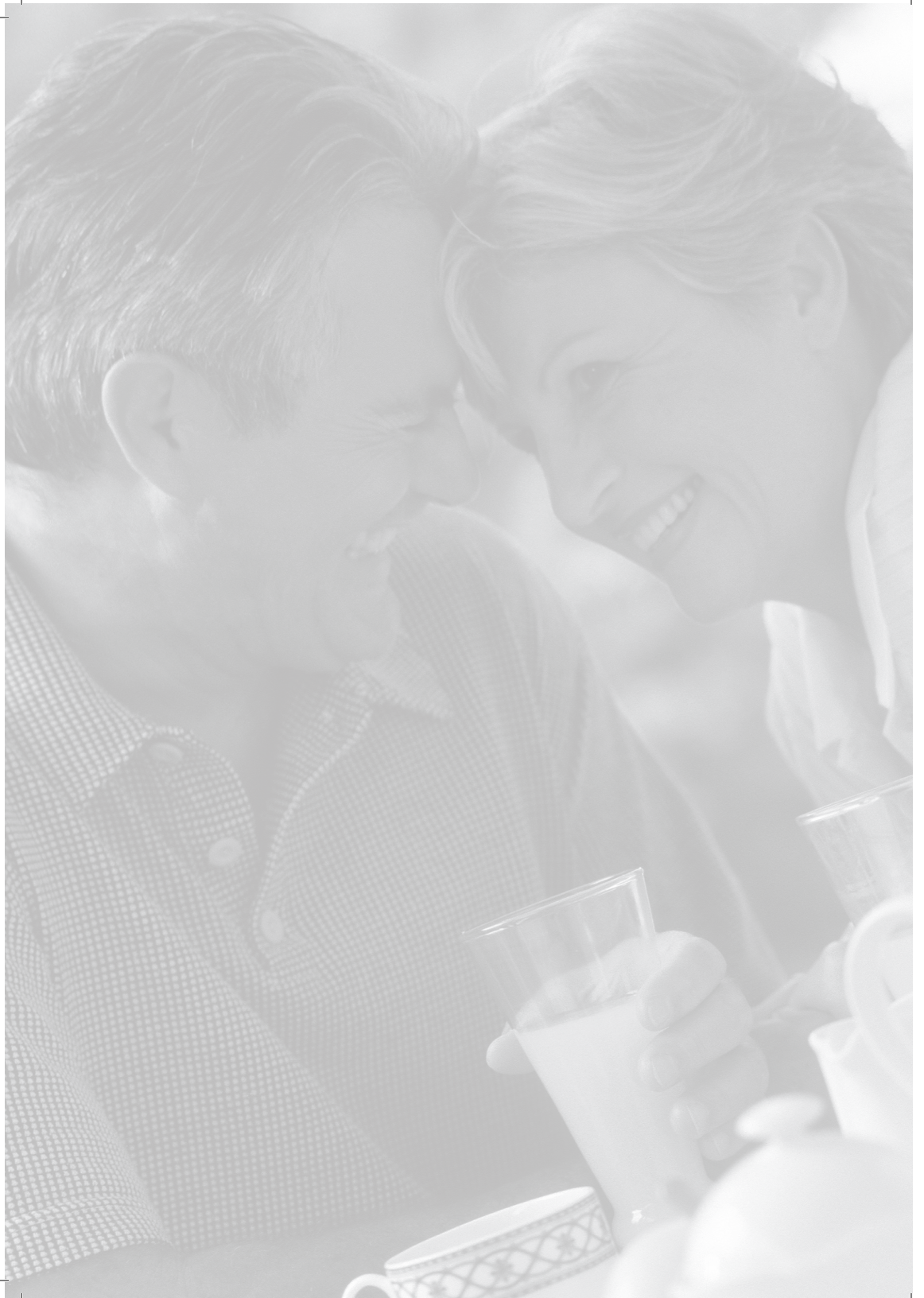
WITH/WITHOUT BACKREST

INTENDED USAGE:

The products of the *Bathtub Seats KING* line act as a bench designed to avoid the use of the bottom of the bathtub, facilitating entrance and exit from it. They reduce the risk of taking a tiring or dangerous position during the sitting and standing procedures. The bathtub seat promotes the user's independence during the bathtub stages of daily personal hygiene. The backrest further increases comfort, offering a more accessible and safer way to bathe. The seat can be completed with a soft cushion to make the use more comfortable. The soft cushions are available separately as accessories.

INDICATIONS AND CONTRAINDICATIONS:

The products of the Bathtub Seats KING line support elderly people, wheelchair users, people with reduced mobility of the lower limbs, lumbar, and hip, and all those who have difficulty lying down and standing up in the bathtub. The ability to sit independently is required to use the bathtub seat. It is not recommended to use the bathtub seat if the user has balance or perception problems.

SAFETY WARNING AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- If the user has mobility problems or suffers from balance disorders, we suggest asking for assistance to enter and exit the bathtub.
- Before each use, the user has to check the integrity of the bathtub seat.
- During the assembly, strictly adhere to what is shown in the User Manual.
- The risk of finger injuries may increase during the width selection step of the assembly stages and the installation process on the bathtub.
- Before every use, be sure the bathtub seat is firmly stable on the bathtub. The end rubber caps and the rectangular rubber strips on the aluminum supports must be correctly leaned to the plane edge and the side of the bathtub respectively.
- Do not modify the bathtub seat and its components in any way.
- Do not use the bathtub seat if damaged or altered in any way.
- During use, avoid positions that keep the body weight too much backward or forward. Wrong positions may lead to instability and overturning of the bathtub seat.
- During the sitting and standing stages, the sliding risk may increase due to the wet floor.
- The bathtub seat must not be stored outdoors.

MAINTENANCE INFO:

Product maintenance consists of cleaning and sanitation procedures. In order to guarantee safe use and an adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components, and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before proceeding with the most accurate disinfection.
3. Use water or non-aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

Alternatively, the product can be cleaned in domestic dishwashers at the maximum temperature. The soft cushion must be cleaned strictly by hand.

REUSE REQUIREMENTS:

The product is suitable for reuse by other users. Before reusing by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the regular household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose of it.

INCIDENT REPORTING:

If there is a serious incident involving this device, contact the manufacturer of this product at info@kingbathrooms.com and the responsible authority in your country.

ITEM CODE	Description	PAGE
KING-BS-KN	Bathtub seat King	44
KING-BSB-KN	Bathtub seat with backrest King	44
KING-BBS-KN	Black bathtub seat King	45
KING-BBSB-KN	Black bathtub seat with backrest King	45
KING-BR-KN	Backrest King	46
KING-BBR-KN	Black backrest King	46
KING-CBS2/4-KN	Soft cushion for bathtub seat light blue/grey King	47

ICON	DESCRIPTION
	Made in Italy
	Easy to assembly
	Dishwasher Friendly
	Max User Weight
	2 Years Warranty
	Scan the QR CODE to watch the assembly video

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LM Global Design Ltd.