 **CANNE STABILICANE**
CANNE STABILICANE
Notice d'utilisation
DISPOSITIF MEDICAL

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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

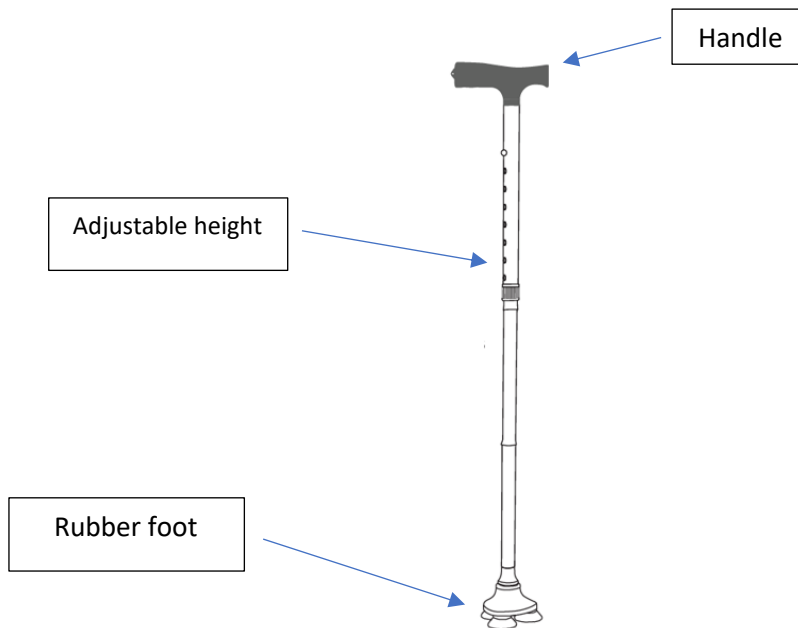
Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

Stabilicane walking sticks are walking aids for people with reduced mobility. They are foldable and easy to transport.

PRODUCT DESCRIPTION

-Foldable design for easy transport and storage with ergonomic handle.

-Cane base features a 'swivel ball joint' design for extra grip and features a 3-rubber foot design for improved stability, ensuring protection from wet surfaces.

- Eight adjustable heights from 77.47 cm to 94.99 cm. Perfect for people between 1.52 and 2.01 meters tall.

SAFETY PRECAUTIONS

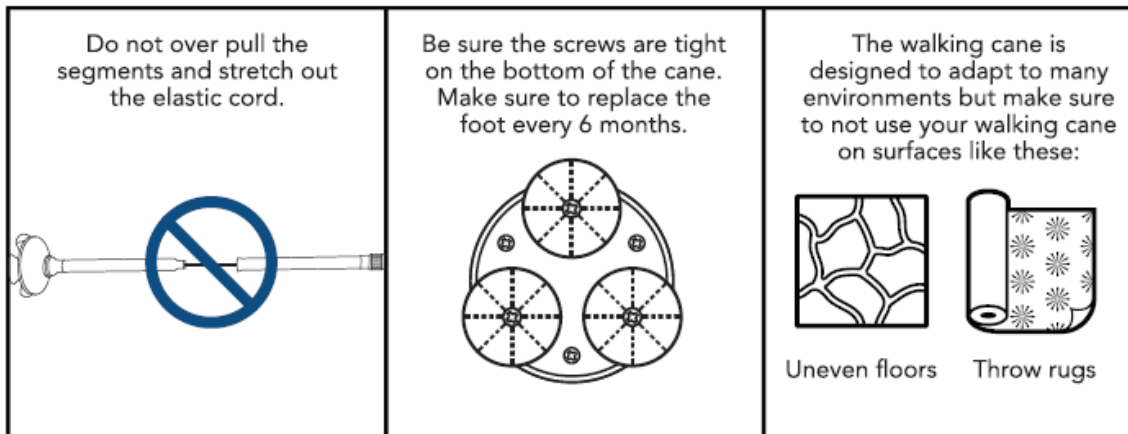
Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 150 kg.
- Use only accessories and parts provided by the manufacturer.
- Visually inspect the overall condition of the product (tube, handle, foot).
- Do not use the walking cane if the cord is damaged; check the elasticity of the cord before each use.
- The walking cane is not intended to fully support a person's weight.
- Ensure that all parts are securely fastened and in good working condition.
- Do not make any adjustments to the product while it is in use.
- Always lean on the handle for support.
- In case of damaged, broken, or worn parts, contact the dealer.

- Consult a physician before using a walking aid.
- Do not use the walking cane unless the segments and push buttons are secure and tightened to lock in place.
- When straightened, keep your fingers away from the segments to avoid pinching.
- Do not stretch the internal elastic cord more than necessary.
- Do not use the walking cane if the elastic cord is cut or damaged.
- Do not use the walking cane if the handle or foot of the cane becomes loose.
- The cane is not designed to be used on slippery or wet surfaces.

WARNING


Do not dismantle the body of the pole. Doing so will damage the elastic cord inside the cane and uncontrolled release of the elastic cord could cause serious injury.


ASSEMBLY AND USE

ASSEMBLY

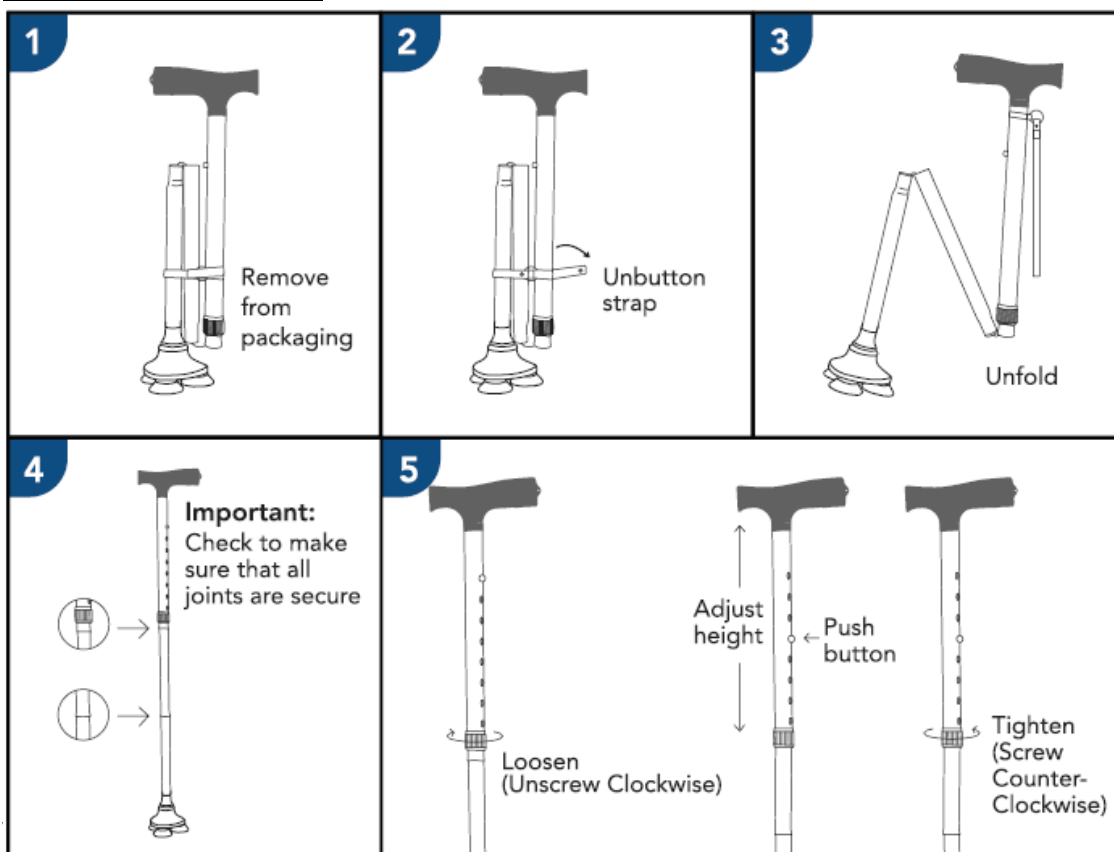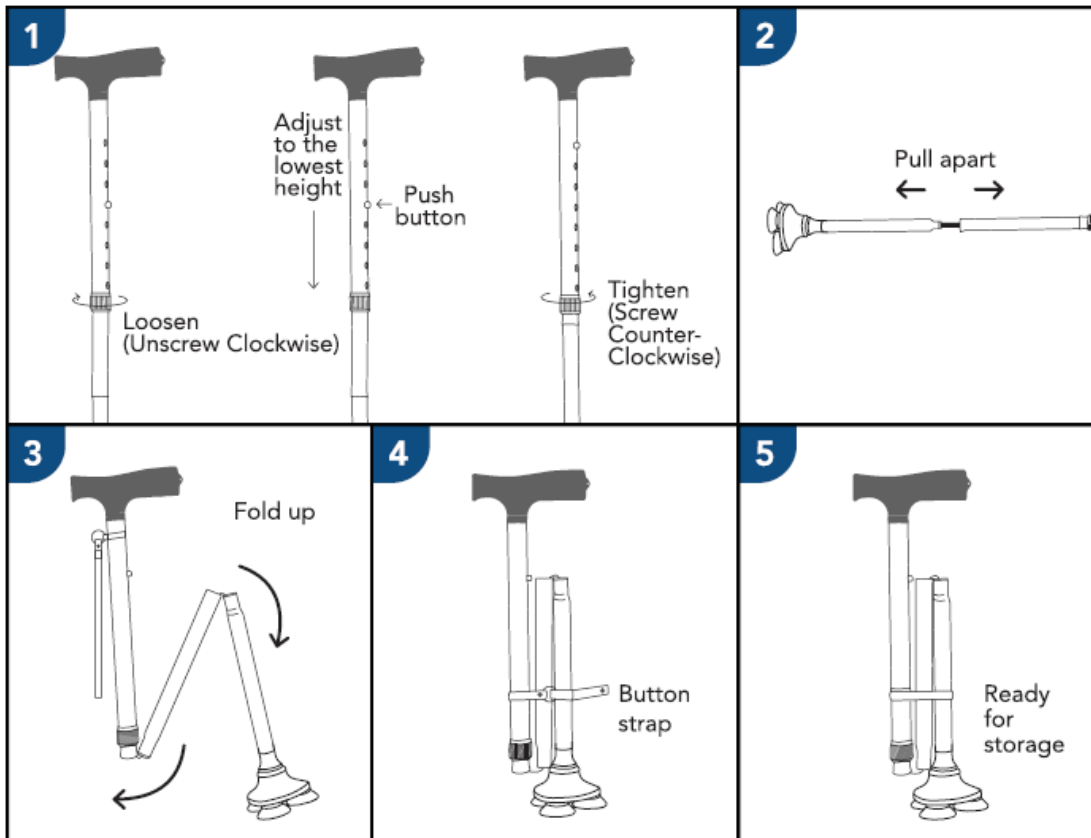
Setting up your walking cane:

USE

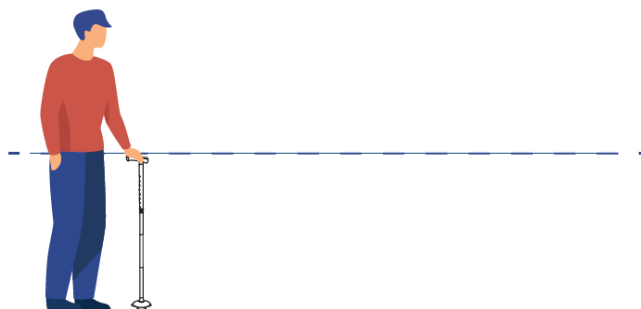To use the product, lean on the handle. Adjust the height if necessary. Move forward slowly. The height of the walking stick can be adjusted between 77 and 94 cm. The best height for your walking stick is the height of your hips.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a soft, clean, dry cloth to prevent oxidation.
- Make sure there is no build-up of dirt or debris around the end caps.
- Check that the screws on the rubber feet are tight; if not, use a screwdriver to tighten the feet for correct use. Replace the rubber feet if they are no longer working properly.
- Do not use cleaning products containing phenol or chlorine, as this could damage the aluminium and polyamide materials.
- Periodically check all fixings, handles and rubber feet for wear and strength.
- Before each use, check that the rubber end cap is in good condition.
- In the event of a malfunction, do not repair the product yourself. Consult an authorised dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable, and you must throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product:
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g., abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

FEATURES TECHNICAL

Over all dimensions	Length 77,47 to 94.99 cm in 8 positions
Maximum supported weight	150 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multiuser environment (such as a nursing home or hospital) then service life may be reduced.

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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